

IVALON

Surgical Products
by **fabco**®

Femoral Canal Sponge

CE 0459

EC REP

MDSS GmbH
Schiffgraben 41
30175 Hannover, Germany

fabco®

Manufactured for:

First Aid Bandage Company • 3 State Pier Road • New London, CT 06320 • USA

Phone: (860) 443-8499 • Fax: (860) 442-8699 • www.fabco.net

Medsorb Dominicana, S.A. • Zona Franca Industrial • San Pedro de Macoris • Dominican Republic

IVALON® is a registered trademark of Medsorb Dominicana S.A.

1391-50369-00116 Rev. D

Femoral Canal Sponge

(Used to absorb fluids during hip replacement surgery)

INSTRUCTIONS FOR USE

Step 1. Before insertion

Clean and suction the femoral canal. Introduce 10 cc of saline into the femoral canal.



Step 2. Insertion

Remove the sponge from the package and insert the unstrung end into the femoral canal. The saline will start the expansion of the sponge within the femoral canal.




Step 3. Expansion

The sponge will quickly absorb the saline and begin to expand to fill the femoral canal. A slight pressure will be exerted against the canal wall.



Step 4. Before removal

Leave the sponge in place for a minimum of one minute. Remove the sponge prior to bone cement application by gently pulling on the drawstring or by grasping the sponge with forceps.

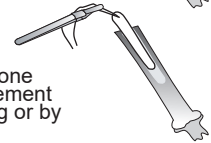

CAUTION

- Federal (USA) law restricts this device to use or sale by or on the order of a physician.
- Sterile. This product is for Single Use Only; do not re-sterilize. Re-use can result in microbial contamination causing subsequent health deterioration of the patient. FABCO will not be responsible for any product that has been re-sterilized or re-used.
- Do not use if individual packaging is damaged or opened. As long as the individual package is not opened, damaged or beyond its expiration date, the product is sterile. Use proper aseptic techniques in all phases of handling.
- Do not cut sponges in close proximity to the surgical procedure; fragments without x-ray detectable abilities may enter the patient's wound site. Failure to remove sponge fragments from the patient may result in a foreign body reaction.

REF

Reorder
number

LOT

Lot
number

STERILE R

Sterilized by
irradiation

⚠

Use
Insert

⊗

Do not
reuse

🕒

Expiration
date

IVALON® is a registered trademark of Medsorb Dominicana S.A.

1391-50369-00116 Rev. D [ENG]