


IVALON

Surgical Products
by **fabco**®

Corneal Light Shield

fabco®

Manufactured for:
First Aid Bandage Company • 3 State Pier Road • New London, CT 06320 • USA
Phone: (860) 443-8499 • Fax: (860) 442-8699 • www.fabco.net

 Medsorb Dominicana, S.A. • Zona Franca Industrial
San Pedro de Macoris • Dominican Republic

IVALON® is a registered trademark of Medsorb Dominicana S.A.

CE₀₄₅₉



MDSS GmbH
Schiffgraben 41
30175 Hannover, Germany

1391-50118-00106 Rev. D

Corneal Light Shield

(For use during ophthalmic procedures to
block excessive illumination)

INSTRUCTIONS FOR USE

Step 1. PRIOR TO USE

Fully hydrate the light shield with saline or sterile water until it softens.

Step 2. PLACEMENT

Grasp the shield with forceps and gently place in the desired location on the cornea. Irrigate as normal.

CAUTION

- Sterile. This product is for Single Use Only; do not re-sterilize. Re-use can result in microbial contamination causing subsequent health deterioration of the patient. FABCO will not be responsible for any product that has been re-sterilized or re-used.
- Do not use if individual packaging is damaged or opened. As long as the individual package is not opened, damaged or beyond its expiration date, the product is sterile. Use proper aseptic techniques in all phases of handling.
- Do not cut sponges in close proximity to the surgical procedure; fragments without x-ray detectable abilities may enter the patient's wound site. Failure to remove sponge fragments from the patient may result in a foreign body reaction.

REF

Reorder
number

LOT

Lot
number

STERILE R

Sterilized by
irradiation



Use
Insert



Do not
reuse



Expiration
date

IVALON® is a registered trademark of Medsorb Dominicana S.A.

1391-50118-00106 Rev. D [ENG]