

IVALON

Surgical Products
by **fabco**®

Eye Drain



MDSS GmbH
Schiffgraben 41
30175 Hannover, Germany

fabco®

Manufactured for:
First Aid Bandage Company • 3 State Pier Road • New London, CT 06320 • USA
Phone: (860) 443-8499 • Fax: (860) 442-8699 • www.fabco.net

Medisorb Dominicana, S.A. • Zona Franca Industrial • San Pedro de Macoris • Dominican Republic

IVALON® is a registered trademark of Medisorb Dominicana S.A.

1391-50117-00105 Rev. D

Eye Drain

(For use in removal and collection of fluids from the
ophthalmic surgical site)

INSTRUCTIONS FOR USE

Step 1. PREPARATION

Thoroughly clean and dry the facial area.

Step 2. COLLECTION BAG APPLICATION


Remove the film from the self-adhesive tape on the collection bag.

(Note: Leave the wick in place within the collection bag.)

Securely attach the bag to the side of the face 1/4" from the eyelid opening. The collection bag should hang down along the side of the face, perpendicular to the forehead. If a drape is used, attach the bag to the drape.

Step 3. POSITION THE WICK

Gently pull the wick from the collection bag to expose the desired length. Moisten the wick with sterile water until the wick softens. Position the wick in the desired location, making sure that the wick is not pinched or obstructed by retractors. Begin the procedure; irrigate as normal.

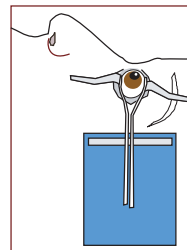

CAUTION

- Moisten wick first. Dry wick may irritate conjunctiva.
- Sterile. This product is for Single Use Only; do not re-sterilize. Re-use can result in microbial contamination causing subsequent health deterioration of the patient. FABCO will not be responsible for any product that has been re-sterilized or re-used.
- Do not use if individual packaging is damaged or opened. As long as the individual package is not opened, damaged or beyond its expiration date, the product is sterile. Use proper aseptic techniques in all phases of handling.
- Do not cut sponges in close proximity to the surgical procedure; fragments without x-ray detectable abilities may enter the patient's wound site. Failure to remove sponge fragments from the patient may result in a foreign body reaction.

REF

Reorder
number

LOT

Lot
number

STERILE R

Sterilized by
irradiation



Use
Insert



Do not
reuse



Expiration
date

IVALON® is a registered trademark of Medisorb Dominicana S.A.

1391-50117-00105 Rev. D [ENG]