

IVALON

Surgical Products
by **fabco**®

Eye Wick



MDSS GmbH
Schiffgraben 41
30175 Hannover, Germany

fabco®

Manufactured for:

First Aid Bandage Company • 3 State Pier Road • New London, CT 06320 • USA
Phone: (860) 443-8499 • Fax: (860) 442-8699 • www.fabco.net

Medsorb Dominicana, S.A. • Zona Franca Industrial • San Pedro de Macoris • Dominican Republic

IVALON® is a registered trademark of Medsorb Dominicana S.A.

1391-50130-00108 Rev. D

Eye Wick

(For use in removal and collection of fluids from the
ophthalmic surgical site)


INSTRUCTIONS FOR USE

Step 1. PREPARATION

Moisten the wick with sterile water until the wick softens

Step 2. POSITION THE WICK

Position the wick in the desired location, making sure that the wick is not pinched or obstructed by retractors. Begin the procedure; irrigate as normal.

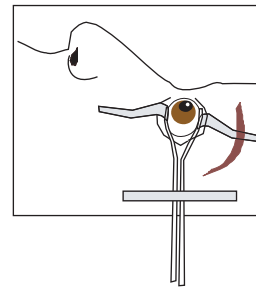

CAUTION

- Moisten wick first. Dry wick may irritate conjunctiva.
- Sterile. This product is for Single Use Only; do not re-sterilize. Re-use can result in microbial contamination causing subsequent health deterioration of the patient. FABCO will not be responsible for any product that has been re-sterilized or re-used.
- Do not use if individual packaging is damaged or opened. As long as the individual package is not opened, damaged or beyond its expiration date, the product is sterile. Use proper aseptic techniques in all phases of handling.
- Do not cut sponges in close proximity to the surgical procedure; fragments without x-ray detectable abilities may enter the patient's wound site. Failure to remove sponge fragments from the patient may result in a foreign body reaction.

REF Reorder number	LOT Lot number	STERILE R Sterilized by irradiation	 Use Insert	 Do not reuse	 Expiration date
---------------------------------	-----------------------------	--	-------------------	---------------------	------------------------

IVALON® is a registered trademark of Medsorb Dominicana S.A.

1391-50130-00108 Rev. D [ENG]