


IVALON

Surgical Products
by fabco®

Instrument Wipe
Paño para instrumentos
Essuyeur d'instrument
Instrumentenwischtuch
Salviettine strumento
Lenço para instrumentos
Instrumentduk
Wycierak do instrumentów
Alet Temizlik Bezi
器械擦拭布

fabco®

Manufactured for:
First Aid Bandage Company • 3 State Pier Road • New London, CT 06320 • USA
Phone: (860) 443-8499 • Fax: (860) 442-8699 • www.fabco.net

 Medsorb Dominicana, S.A. • Zona Franca Industrial
San Pedro de Macorís • Dominican Republic

IVALON® is a registered trademark of Medsorb Dominicana S.A.

CE 0459



MDSS GmbH
Schiffgraben 41
30175 Hannover, Germany

1391-50129-00114 Rev. E

Instrument Wipe

(For use in cleaning debris from delicate surgical instruments)

INSTRUCTIONS FOR USE

Step 1. PREPARATION

Moisten one half of wipe with sterile water or saline prior to use.

CAUTION

- Sterile. This product is for Single Use Only; do not re-sterilize. Re-use can result in microbial contamination causing subsequent health deterioration of the patient. FABCO will not be responsible for any product that has been re-sterilized or re-used.
- Do not use if individual packaging is damaged or opened. As long as the individual package is not opened, damaged or beyond its expiration date, the product is sterile. Use proper aseptic techniques in all phases of handling.
- Do not cut sponges in close proximity to the surgical procedure; fragments without x-ray detectable abilities may enter the patient's wound site. Failure to remove sponge fragments from the patient may result in a foreign body reaction.

REF

Reorder
number

LOT

Lot
number

STERILE R

Sterilized by
irradiation



Use
Insert



Do not
reuse



Expiration
date

IVALON® is a registered trademark of Medsorb Dominicana S.A.

1391-50129-00114 Rev. E [ENG]