

IVALON

Surgical Products
by fabco®

PVA Knife Cleaning Block



MDSS GmbH
Schiffgraben 41
30175 Hannover, Germany

fabco®

Manufactured for:
First Aid Bandage Company • 3 State Pier Road • New London, CT 06320 • USA
Phone: (860) 443-8499 • Fax: (860) 442-8699 • www.fabco.net

 Medsorb Dominicana, S.A. • Zona Franca Industrial • San Pedro de Macoris • Dominican Republic

IVALON® is a registered trademark of Medsorb Dominicana S.A.

1391-50120-00115 Rev. D

PVA Knife Cleaning Block

INSTRUCTIONS FOR USE

Step 1. PRIOR TO USE

Hydrate one end of the cleaning block with a few drops of sterile water. (Please note that the IVALON® Cleaning Block rapidly absorbs fluids - only a few drops are necessary. Do not fully saturate the block.

Step 2. TO CLEAN A KNIFE

Gently wipe knife blade on the wet end of the cleaning block. Repeat this procedure several times. Then stab the dry end several times before sterilization.

NOTE: For optimum performance, use the wet end of the cleaning block to wash instruments and the dry end to remove stuck on particles.



CAUTION

- Sterile - Single use only. Re-use can result in microbial contamination causing subsequent health deterioration of patient. FABCO will not be responsible for any product that has been re-sterilized and/or re-used.
- Do not cut knife blocks or allow fragments in close proximity to the surgical procedure. Sponge fragments may enter the patient's wound site. Failure to remove sponge fragments from the patient may result in foreign body reaction.
- Use of device must be conducted by a physician.
- Do not use if packet is opened, damaged or beyond expiration date to prevent risk of contamination and loss of sterility. Use proper aseptic techniques in all phases of handling.
- Do not exert excessive pressure on the knife blade.
- Used knife cleaning blocks should be considered as bio waste and be disposed of in accordance with local regulations.

REF

Reorder
number

LOT

Lot
number

STERILE R

Sterilized by
irradiation



Use
Insert



Do not
reuse



Expiration
date

IVALON® is a registered trademark of Medsorb Dominicana S.A.